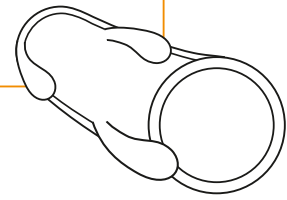
# RESORBA® Barbed Suture

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- *siliconised needles made of 300 steel*
- *synthetic with a long period of action*
- *unidirectional spiral angle cut on one side*
- *high tensile strength*
- *triangular end stopper*

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RESORBA® Barbed Suture is a polydioxanone knotless tissue-closure device, which is comprised of barbed dyed polydioxanone (PDO) suture material, armed with a surgical needle on one end and end stopper at the other end. The RESORBA® Barbed Suture barbs are unidirectional oriented to allow for tissue approximation without the need to tie surgical knots.

RESORBA® Barbed Suture is consisting of polyester, poly(p-dioxanone) and is dyed with D&C violet No. 2. While the formation of barbs in the RESORBA® Barbed Suture reduces the tensile strength relative to non-barbed suture material of the same size, tying of knots in non-barbed suture materials also reduces their effective strength. For this reason, the strength of the RESORBA®

Barbed Suture can be compared to USP knot strength of non-barbed sutures. After implantation the breaking load is at 55% after 42 days. Absorption by hydrolysis is completed after about 180 – 220 days.

- ➔ Colour: violet
- ➔ Chemical name: poly (p-dioxanone)
- ➔ Thread diameter: USP 4-0 -1 (1.5-4 metric)
- ➔ Types of packaging: needle-thread-combinations
- ➔ Sterilization method: ethylene oxide

USP strength before barbed suture	USP strength of RESORBA® Barbed Suture	Number of barbs per inch
2	1	32
1	0	33
0	2-0	42
2-0	3-0	51
3-0	4-0	54

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## Properties of RESORBA® Barbed Suture

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### Needle

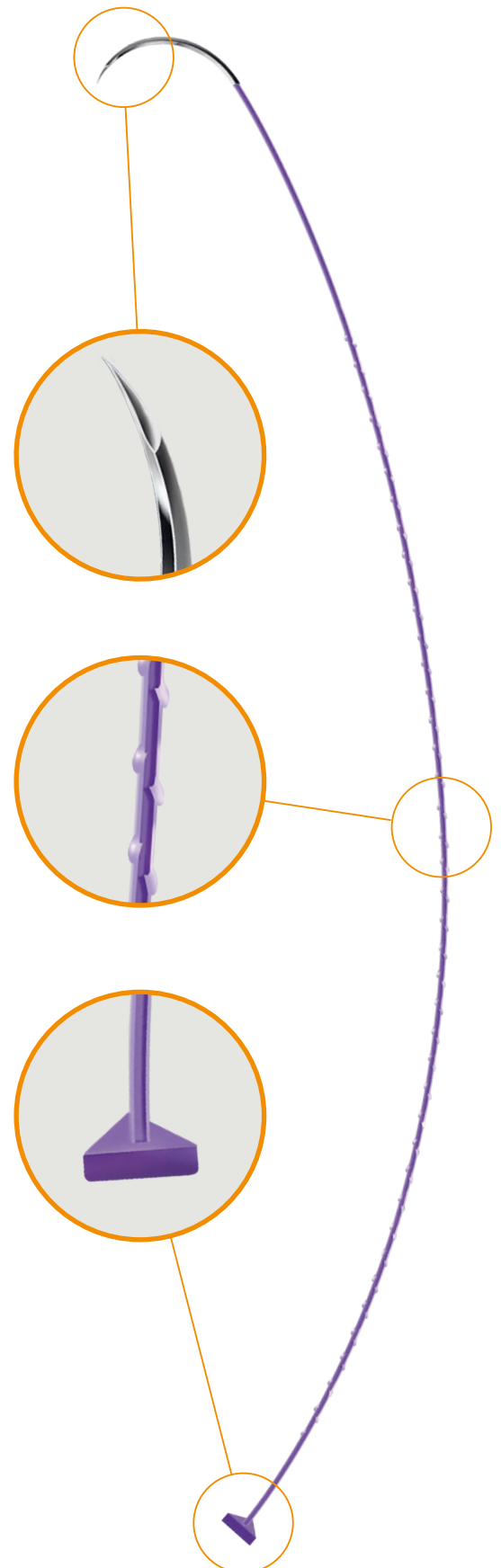
*300 series steel needle provides high bending stiffness and surface siliconisation improves glide through the tissue.*

### Barbed Suture

*The number of **unidirectional barbs** is adapted to the respective thread strength. The optimized distance between the individual barbs and their special shape ensure secure and firm fixation across all thread sizes.*

### End stopper

*The triangular end stopper eliminates the need for "looping" at the beginning of suturing, making handling quicker and easier. The end stopper is perpendicular to the tissue and thus prevents the suture from running out and fixes the suture in the desired position.*

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