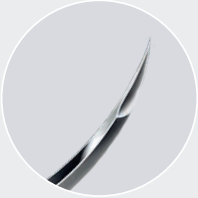
RESORBA® Barbed Suture

RESORBIERBARES NAHTMATERIAL

RESORBA®
SUTURES & SUPPLIES

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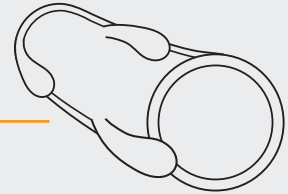
- ✓ *siliconised needles made of 300 steel*
- ✓ *synthetic with a long period of action*
- ✓ *unidirectional spiral angle cut on one side*
- ✓ *high tensile strength*
- ✓ *triangular end stopper*

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► RESORBA® Barbed Suture (PDO) is a polydioxanone knotless tissue-closure device, which is comprised of barbed dyed polydioxanone (PDO) suture material, armed with a surgical needle on one end and an end stopper at the other end. RESORBA® Barbed Suture (PDO) features unidirectionally oriented barbs that facilitate tissue approximation without the need for tying surgical knots. Although the barbs in the RESORBA® Barbed Suture (PDO) reduce tensile strength compared to non-barbed

suture material of the same size, tying knots in non-barbed sutures also diminishes their effective strength. For this reason, the strength of the RESORBA® Barbed Suture (PDO) can be compared to USP knot strength of non-barbed sutures. After implantation, the tensile strength remains at 55 % after 42 days. Absorption by hydrolysis is completed after about 180 to 220 days. RESORBA® Barbed Suture (PDO) is available in violet (coloured with the physiologically harmless D+C No. 2 dye).

USP strength before barbed suture	USP strength of RESORBA® Barbed Suture	Number of barbs per inch
2	1	32
1	0	33
0	2-0	42
2-0	3-0	51
3-0	4-0	54

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Properties

- ✓ siliconised needles made of 300 steel
- ✓ synthetic with a long period of action
- ✓ unidirectional spiral angle cut on one side
- ✓ high tensile strength
- ✓ triangular end stopper

- **Colour:** violet
- **Chemical name:** poly(p-dioxanone)
- **Thread diameter:** USP 4-0 - 1 (1.5-4 metric)
- **Types of packaging:** needle-thread-combinations
- **Sterilization method:** ethylene oxide

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Needle

300 series steel needle provides high bending stiffness and surface siliconisation improves glide through the tissue.

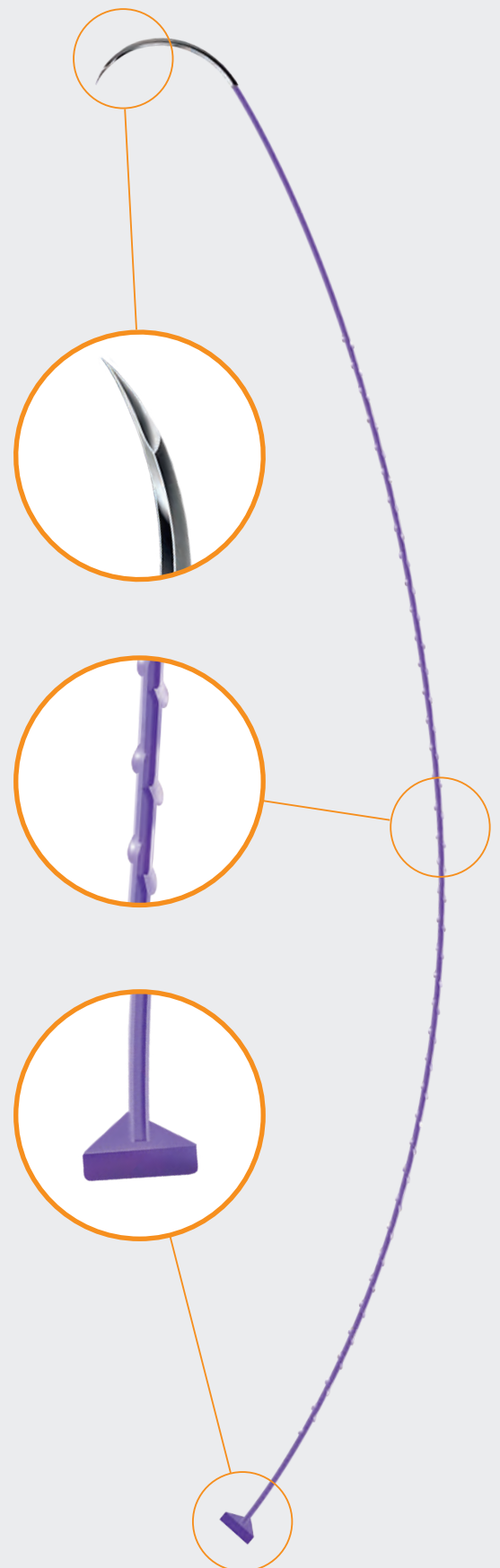
Barbed Suture

The number of **unidirectional barbs** is adapted to the respective thread strength. The optimized distance between the individual barbs and their special shape ensure secure and firm fixation across all thread sizes.

End stopper

The triangular end stopper eliminates the need for "looping" at the beginning of suturing, making handling quicker and easier.

The end stopper is perpendicular to the tissue and thus prevents the suture from running out and fixes the suture in the desired position.

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RESORBA Medical GmbH